



SafetyNET[®]
Guidewire Introducer

FRONTLINE MEDICAL PRODUCTS
 990 Ironwood Drive, Suite 200, MINDEN, NV 89423
 (775) 267-9588.

www.frontlinemedical.com

THE SAFETY ENDOVASCULAR ACCESS NEEDLE DEVELOPED WITH CLINICIANS IN MIND

The SafetyNET[®] Guidewire Introducer was designed with Clinicians for Clinicians. Our dual patented **NO EXPOSURE TIME[™]**, OSHA-compliant safety-engineered needle is designed to provide “familiarity of feel,” and unparalleled control & protection.

When introducing guidewires, Clinicians rely on ultrasound to successfully access the vasculature. Our new **iECHO[®]** feature surrounds the distal end of the needle tip, giving you the ability to precisely locate the needle tip itself. Our **iSHARP[™]** needle tip is designed to provide superior sharpness for navigating through tough scar tissue and endoluminal calcification. The Clinician-designed **Access Control[™]** grip is specifically designed to provide improved control whether inserting or withdrawing the needle.

The SafetyNET is activated by a single hand, allowing the non-dominant hand to be free to manipulate the guidewire & control patient bleeding. Once the safety feature is activated, blood in the sharp needle tip is contained within the Safety Guard as it instantly captures the needle tip before cross-contamination can occur, protecting caregivers and their patients while simultaneously providing the “familiarity of feel” Clinician’s rely on for successful vascular access.

SafetyNET[®] Features & Benefits

Patented Dual Feature Safety Guard

Designed to:


- Instantly capture sharp needle tip on activation;
- Protect caregivers from cross-contamination;
- Contain blood within the Safety Guard.

Provides critical “familiarity of feel”

Designed to be activated with a single hand

Manipulate guidewire and control bleeding with the free hand

Designed with Clinicians for Clinicians

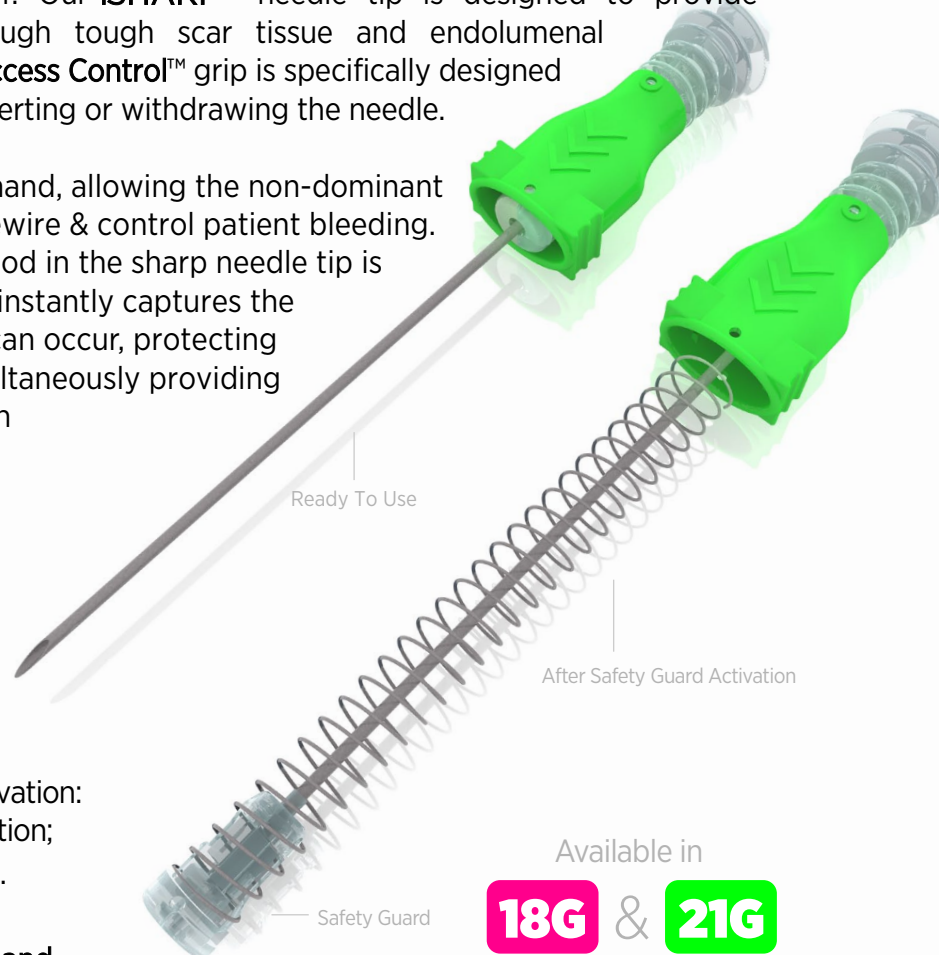


When Outcomes Matter sm - Total tip containment. No Exposure Time. SafetyNET[®]



SafetyNET[®]

Guidewire Introducer

1.

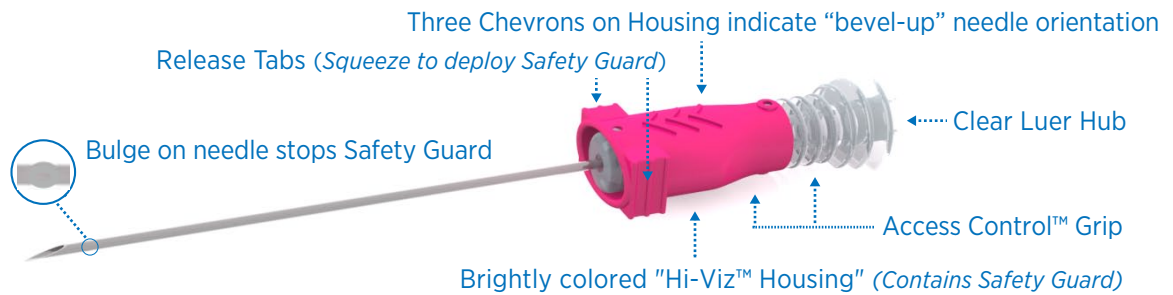


SafetyNET[®] Guidewire Introducer includes our Access Control grip at the Luer Hub/Housing interface, for securely holding the Introducer needle during vascular access and guidewire placement. The Safety Guard will be deployed at the user's discretion by concentrically squeezing the Release Tabs. Hi-Viz Housing helps you locate the used needle during clean up.

2.




Guidewire has been advanced to desired location, and the SafetyNET[®] has been withdrawn from patient, but remains positioned on the guidewire. The Safety Guard is now ready to be deployed. With the dominant hand, concentrically squeeze the Release Tabs on the Housing to activate the Safety Guard.

3.

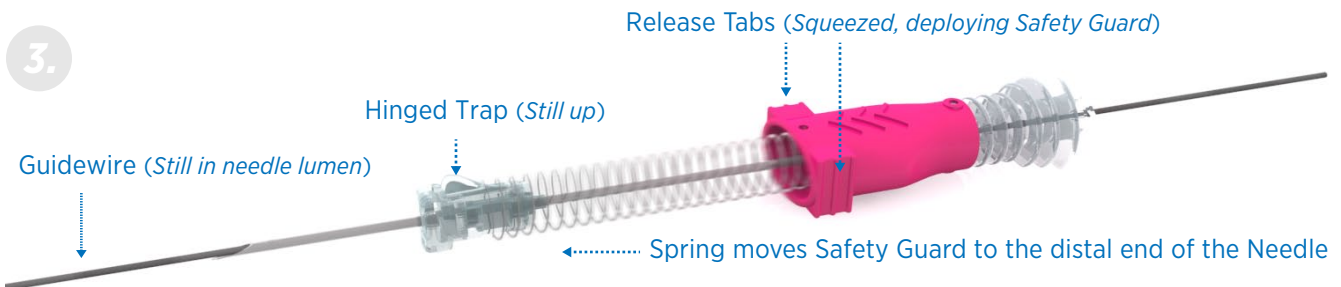

Safety Guard has been deployed after squeezing the Release Tabs on the Housing. One-handed activation permits the non-dominant hand to stabilize guidewire and control bleeding. Needle is still on the guidewire. Safety Guard is moved toward the end of needle by the spring. Hinged Needle Trap in Safety Guard is still up sliding on the needle shaft.

4.

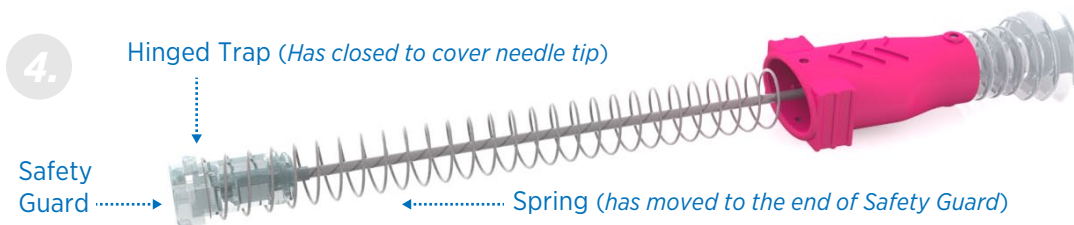

Needle has been removed completely off the guidewire. Hinged Needle Trap in Safety Guard closes, capturing needle tip within internal blood containment chamber. Spring has moved forward on Safety Guard to lock Needle Trap in place. Deployment of the SafetyNET, THINLINE, and iECHO are registered U.S. trademarks of InjectiMed, Inc. FRONTLINE MEDICAL PRODUCTS, ACCESS CONTROL, ISHARP, WHEN OUTCOMES MATTER and NO EXPOSURE TIME are tradenames of InjectiMed, Inc.

Frontline Medical Products: 990 Ironwood Dr., Suite 200, Minden, NV 89423 (775) 267-9588 www.frontlinemedical.com
For sales of sterile kits, please contact Jose Perez at Medica International: (646) 599-5436 or email jose@medicaintl.net

U.S. Patents: #8,672,895; #8,821,439; & #9,610,403. Foreign Patents: AU 2018200868; CA 2904919; CA 2881423; JP 6793410 and Patent Pending. SafetyNET, THINLINE, and iECHO are registered U.S. trademarks of InjectiMed, Inc. FRONTLINE MEDICAL PRODUCTS, ACCESS CONTROL, ISHARP, WHEN OUTCOMES MATTER and NO EXPOSURE TIME are tradenames of InjectiMed, Inc.